




Attest™
Sterilization Solutions

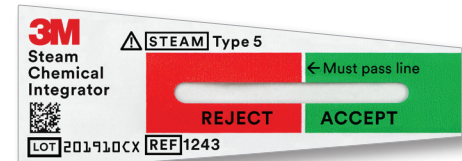

3M™ Attest™ Steam Chemical Integrator Guide

Recommended Placement


Wrapped Towel Pack or Gown, Towel and Drape Pack

Place CI in geometric center, not on top.




Manufacturer Supplied Container (multi-level, perforated tray, needs to be wrapped)*

Place CIs in center on each level.

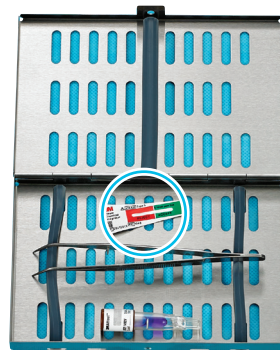


Wrapped Instrument Tray

Place CI in geometric center, not on top.

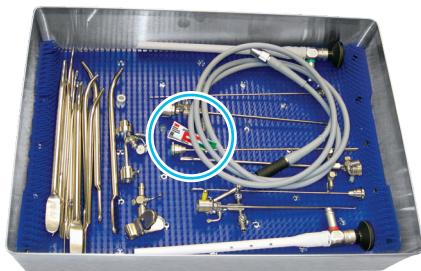

Rigid Containers*

Place two CIs in opposite corners of each level.

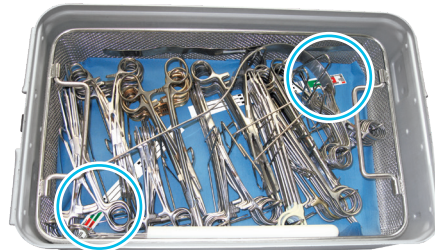


Peel Pouches

When an internal CI is visible, an external Type 1 CI is not required.



IUSS Sterilization of Rigid Containers*

Place two CIs in opposite corners.

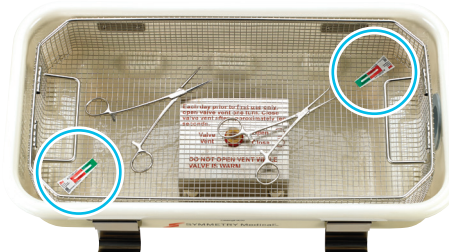

Contact your 3M Representative, call the 3M Health Care Help Line at **1-800-228-3957** or **go.3M.com/steam**


3M Health Care
3M Medical Solutions Division
3M Center, Building 275-4E-01
St. Paul, MN 55144 USA
1-800-228-3957
3M.com/medical

*The manufacturers of the device and container system should also be consulted for recommendation on number and placement of internal chemical indicators.

© 2020 3M. All rights reserved. 3M and Attest are marks and/or registered marks of 3M. Unauthorized use prohibited. Please recycle. Printed in USA. 70-2010-7298-3


Interpretation

Unprocessed

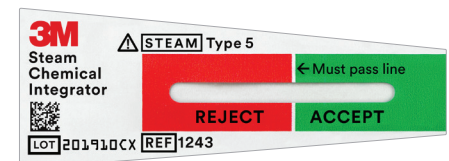


Processed — ACCEPT

If the color bar reaches or crosses into the ACCEPT window, it is a pass — necessary conditions for sterilization have been met.

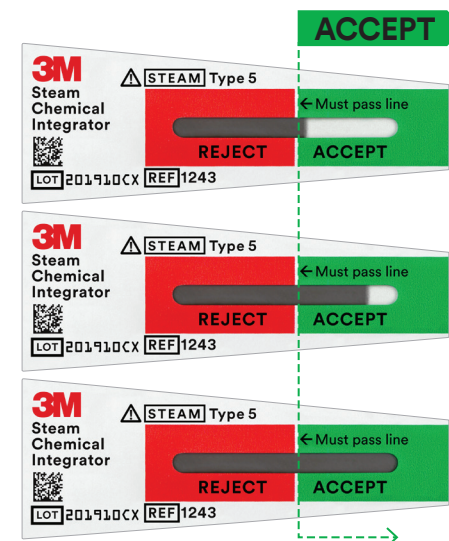

Processed — REJECT

If the color bar is in the REJECT region or on the line, it is considered a fail. The pack should be reprocessed and the cause of sterilization failure should be investigated.

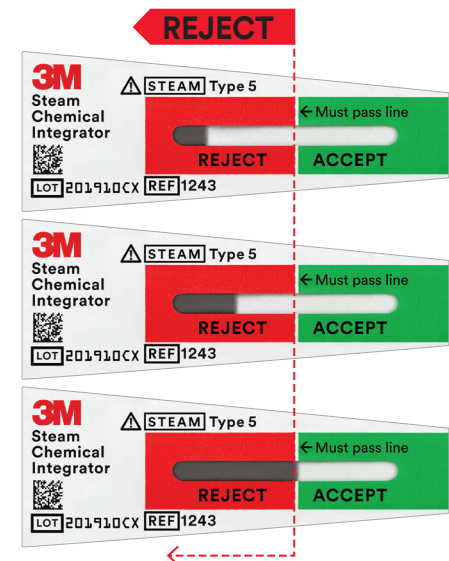

ANSI/AAMI ST9:2017 Internal CIs should be placed
a) so that one CI is visible to the person opening the package;
b) in the area or areas considered least accessible to steam penetration; and
c) in accordance with all applicable written IFUs

3M™ Attest™ Steam Chemical Integrators, Type 5 conform with ISO 11140-1:2014.