

KIMTECH PURE* A5 Integrated Hood and Mask Donning Technique



Step 1

Select your integrated hood and mask;
inspect package for vacuum seal; tear
along notches toward the product to open



Step 2

Pull out integrated hood and unfold
to locate the blue signal indicator line



Step 3

Center your chin just above the blue
line then roll over your forehead.



Step 4

Grab ties with both hands then extend
arms out in a circular motion to position
under armpits.



Step 5

Pull ties across chest and secure as needed



Step 6

Gently adjust front flap to comfortable
position; pinch mask wire over bridge
of nose to ensure proper fit.

